



HIGHTOP - SARS-CoV-2

Rapid Antigen Test Programme

Programme components;

1. Very High Sensitivity Rapid Antigen Test
2. Mobile Digital App
3. Management Reporting
4. Consumer Test Support
5. Testing Programme Support Manager


ARTG ID: 385179



Product Overview

HIGHTOP Antigen Rapid Test for the detection of SARS-CoV-2 antigens in samples from the human anterior nasal cavity area. It is used to detect SARS-CoV-2 nucleoprotein antigens within 7 days of the onset of symptoms suspected of coronavirus infection.

Intended Use: For self-testing
Type: Anterior nasal (2-2.5cm)

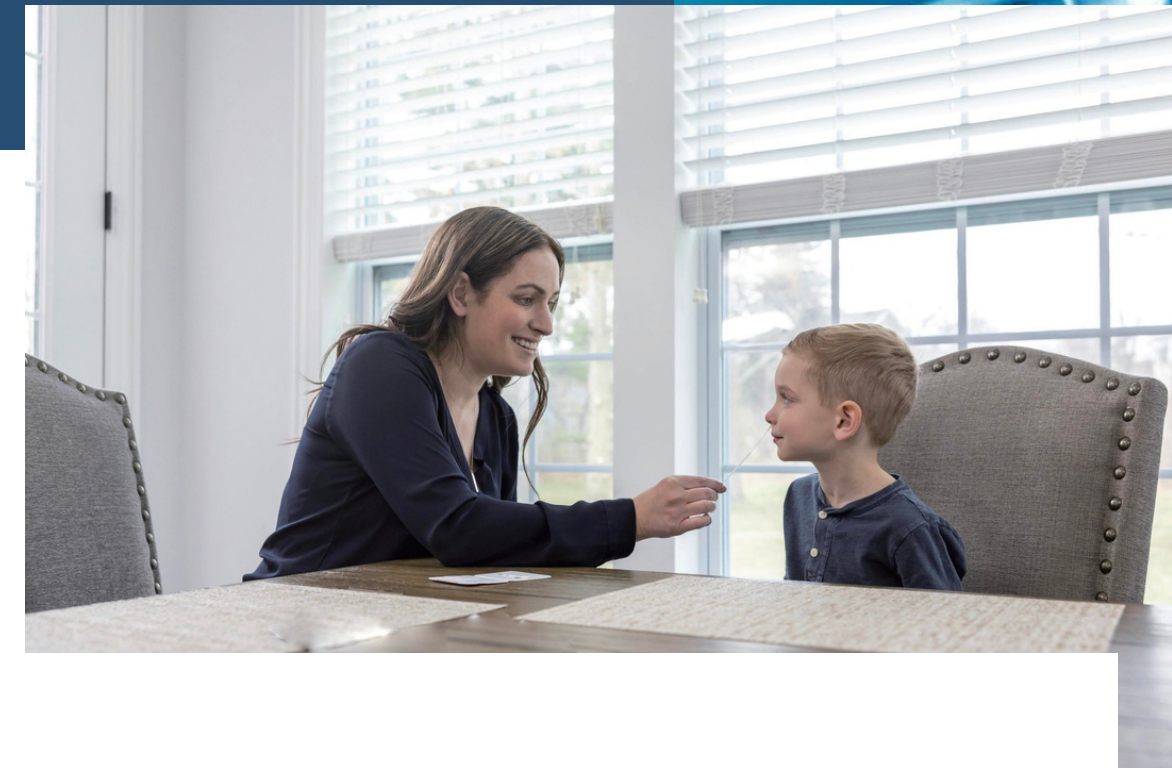

Product Characteristics

Clinical Performance:

- Sensitivity: 98.04%
- Specificity: 100%
- Limit of Detection (LoD): 8 TCID 50/ml

Others:

- Nasal test
- Result in 15 min
- Suitable for Adults and Children 2+ years (with supervision)
- Sold so far in over 40 countries (CE-Certification)



Kit Composition




Test Cassette



Extraction Tube
(pre-filled)






Nasal Swab

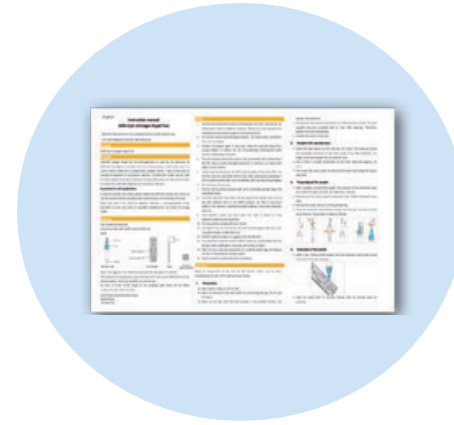

IFU & SOP








Kit size: Tests per box: 1 or 5 or 25 available

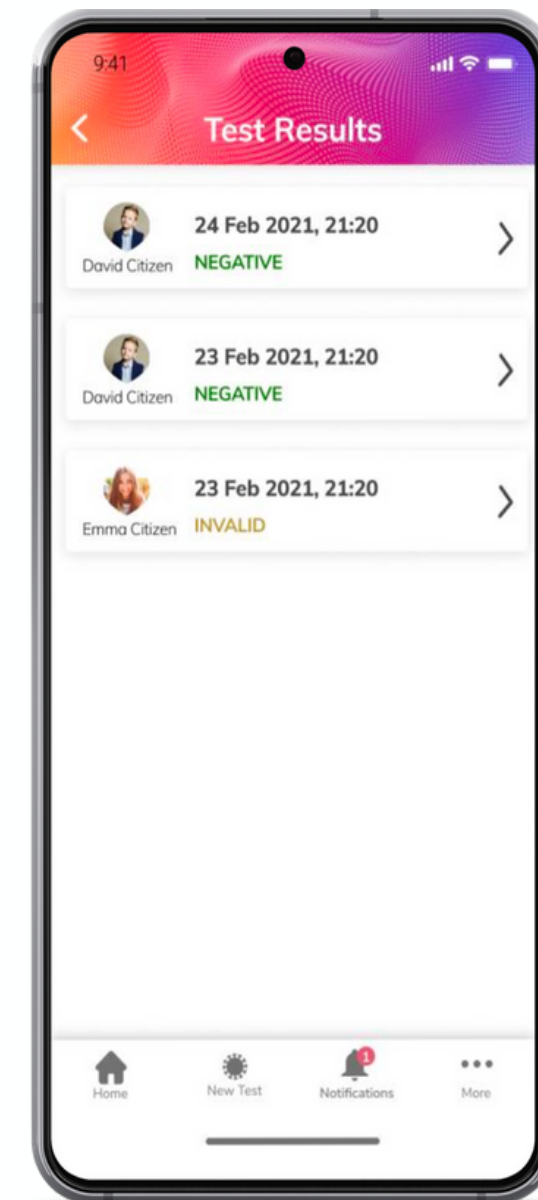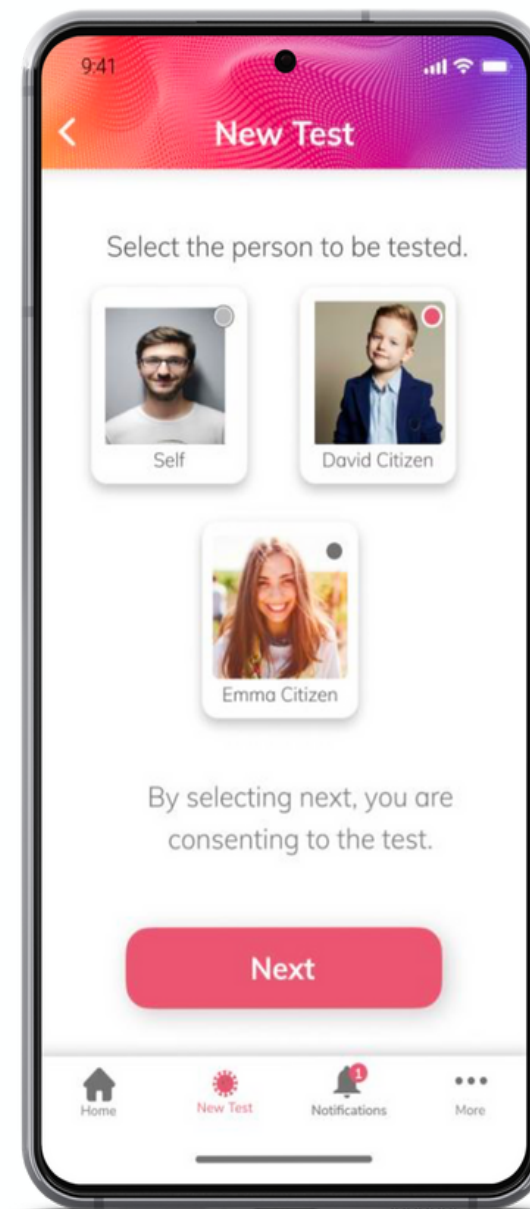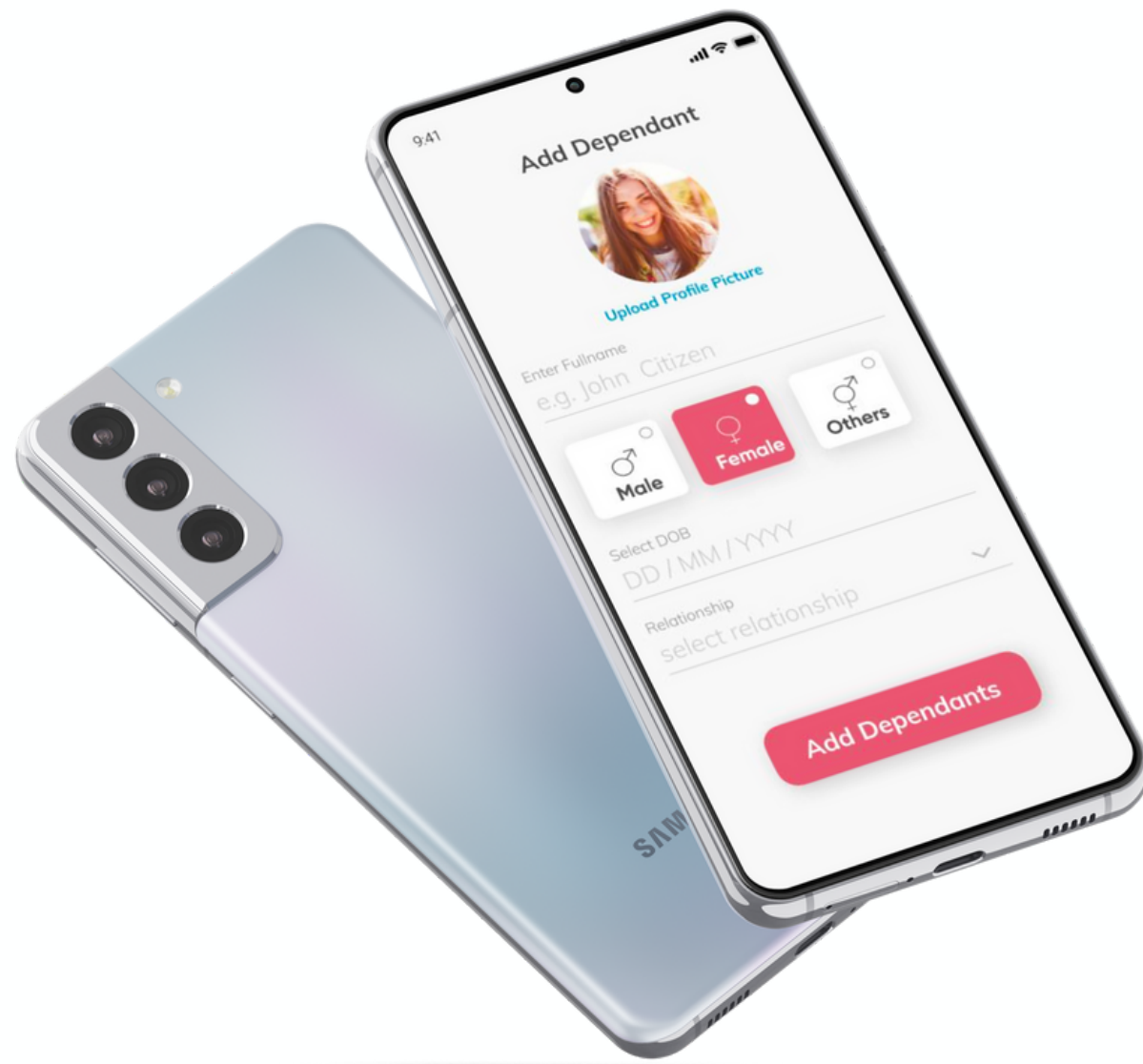

SELFHECK APP

Easy to use for individuals, dependents and families.

Our software app is intuitive and well tested, with numerous improvements and enhancements since launch. This is the **GOLD** standard for easy user experience for all the family.

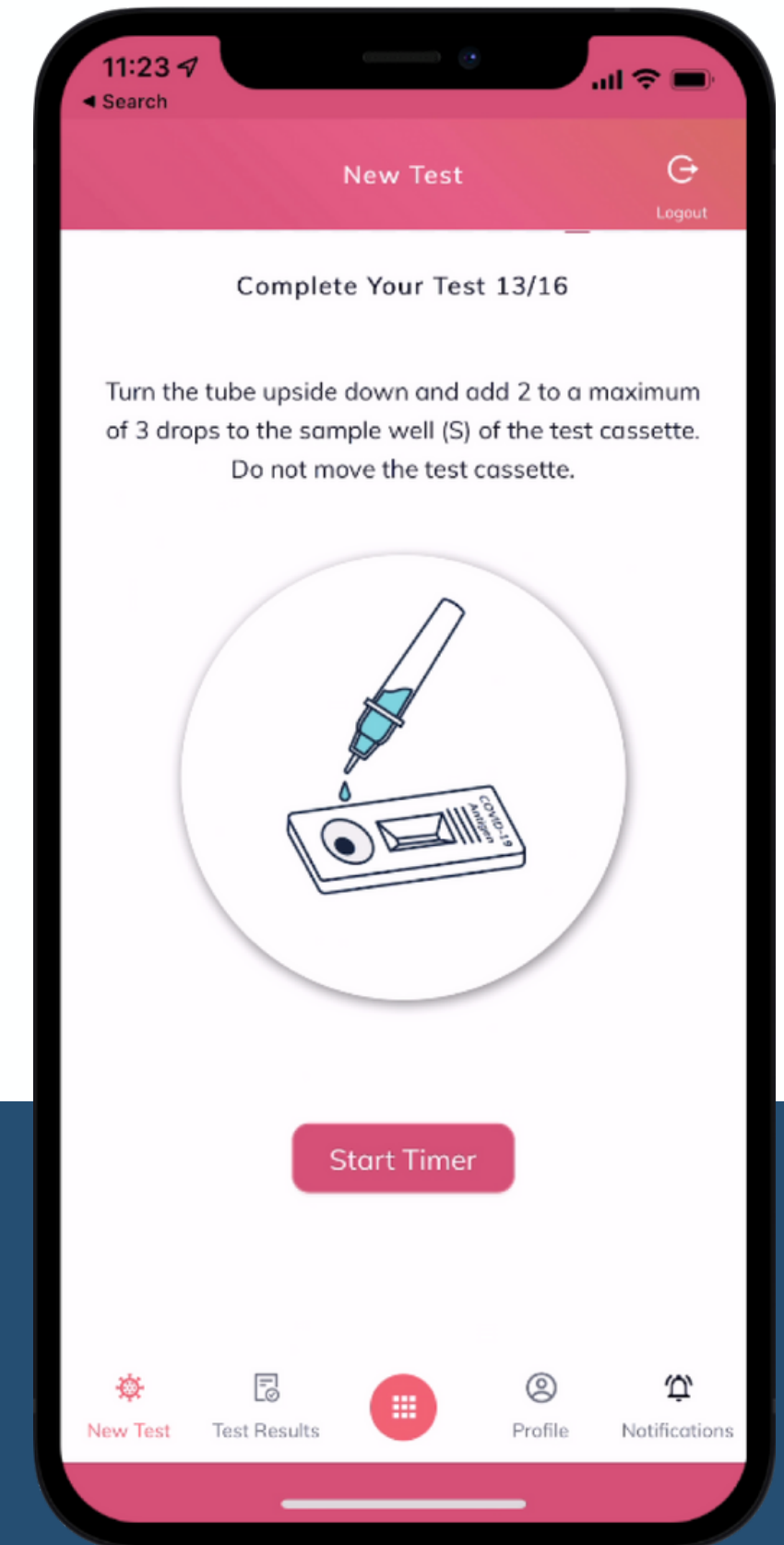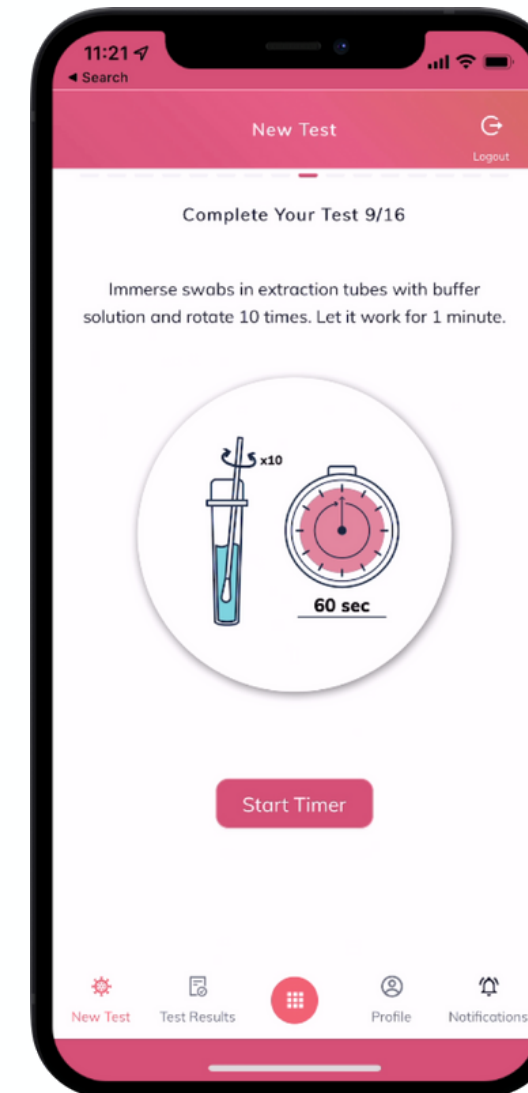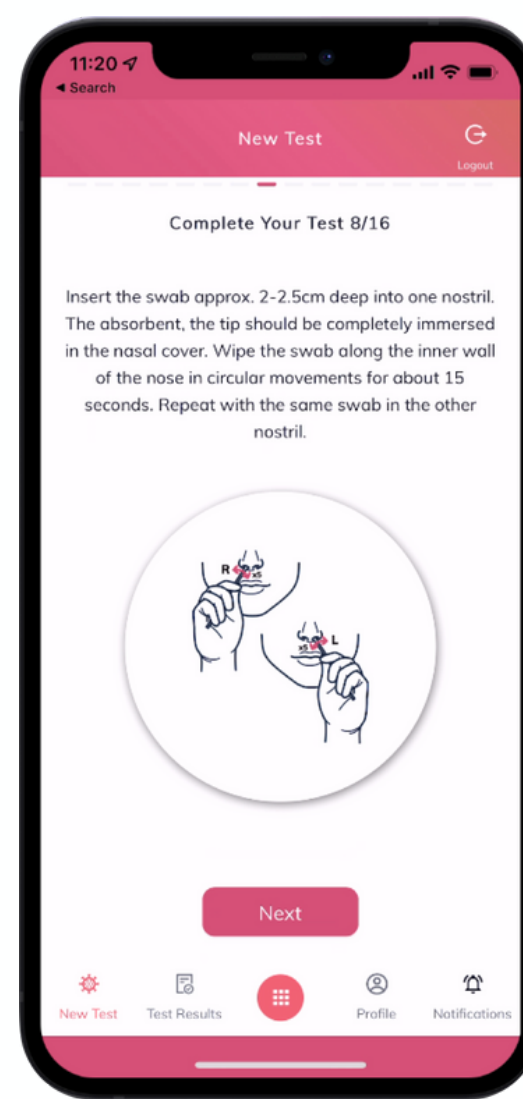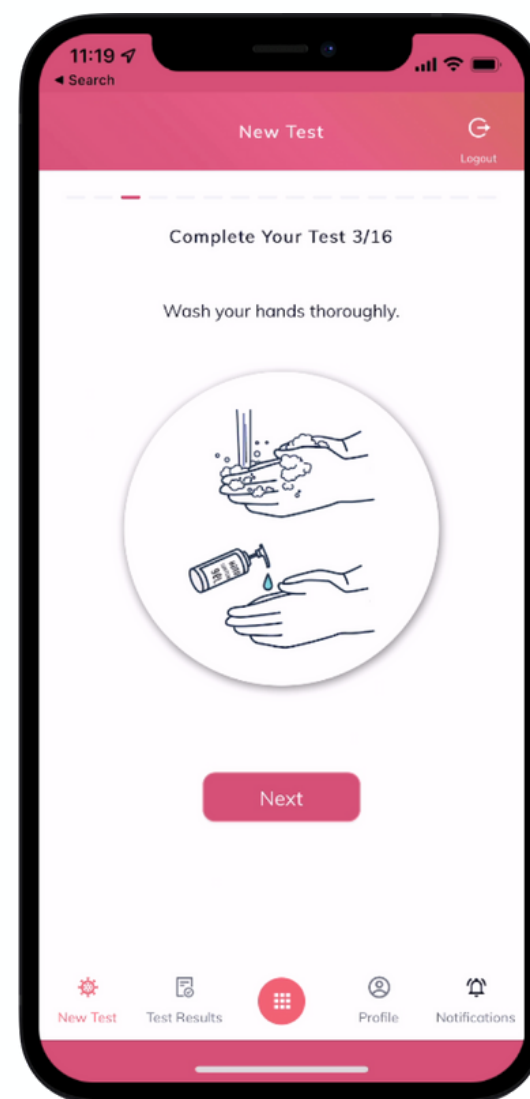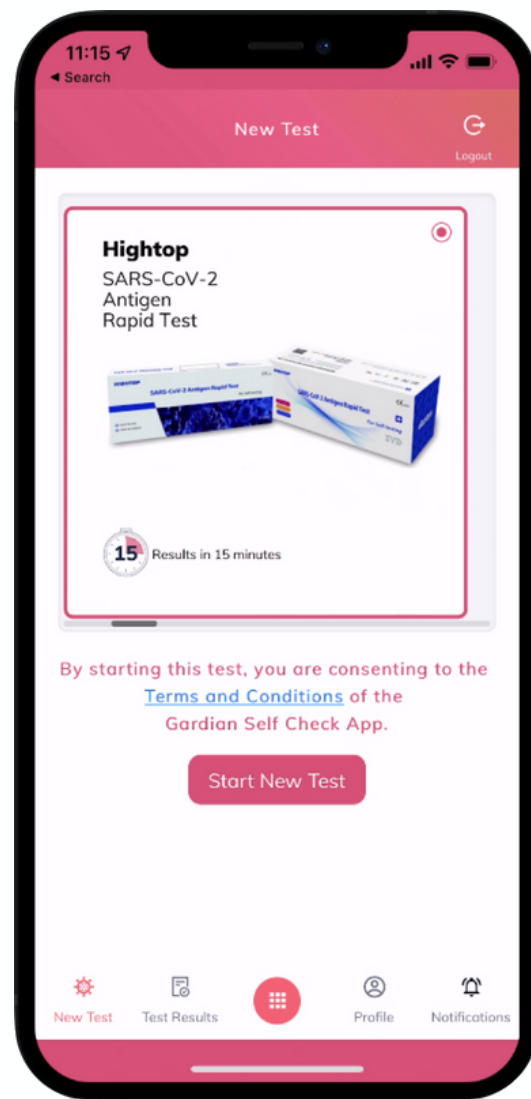


EASY TO FOLLOW TEST PROCEDURE

Our step-by-step digital user guides provide simple explanation of the test procedure at every step.


We provide a full video demo and intro as part of the software demo.




Interpretation of Results

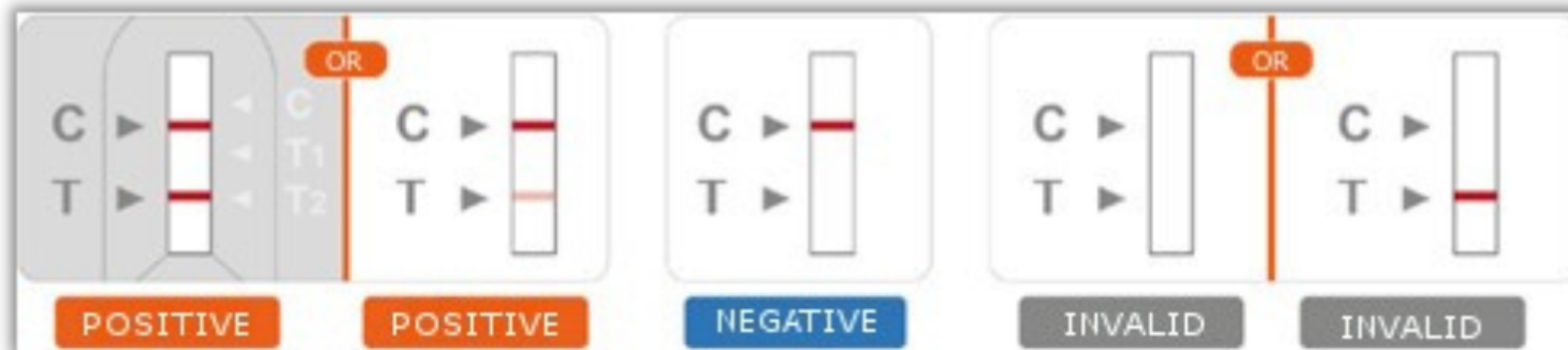

Clear, easy-to-read guide to provide an accurate determination of the result.

This is present on the paper IFU and in the **SelfCheck** Mobile App

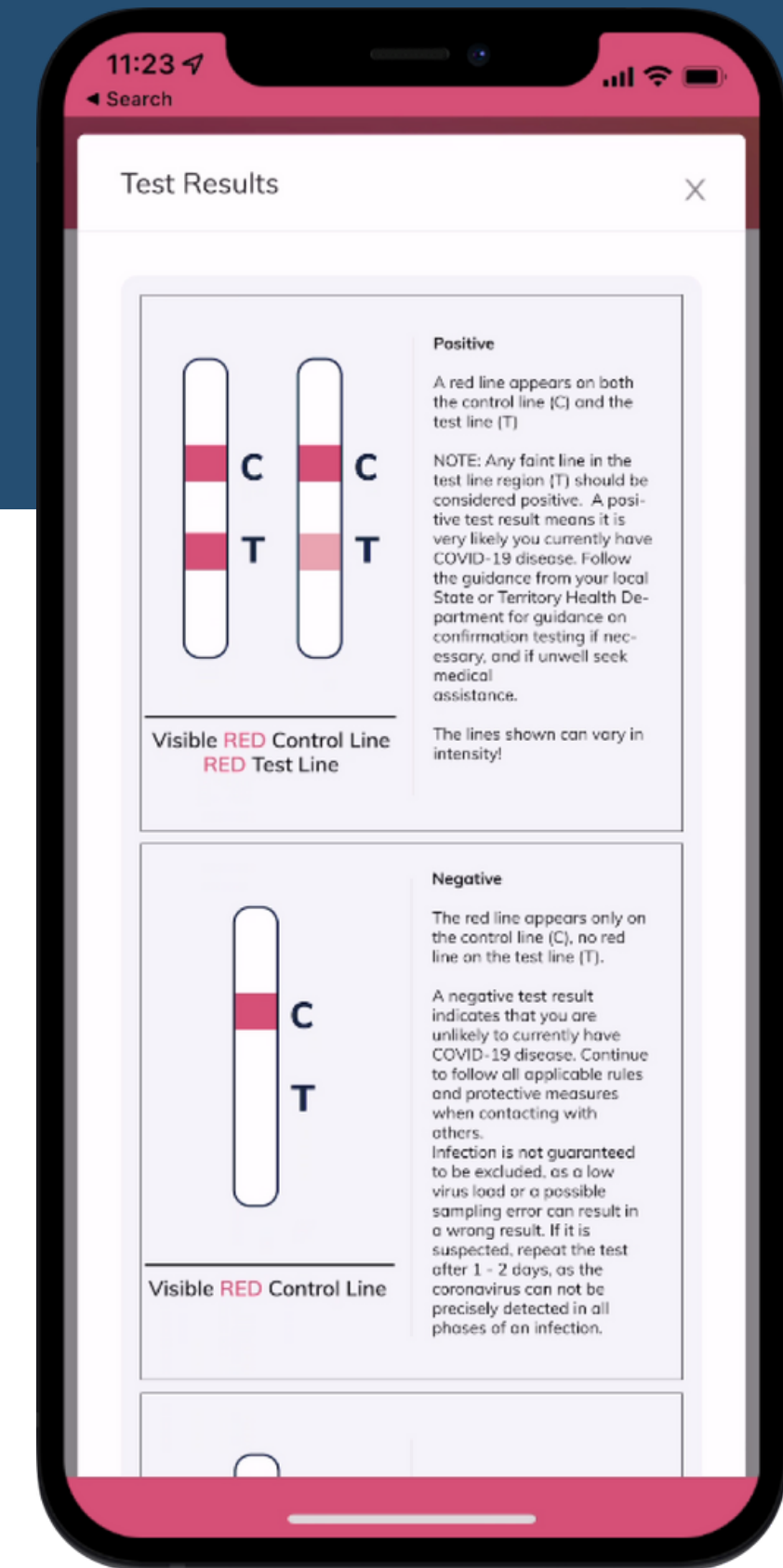

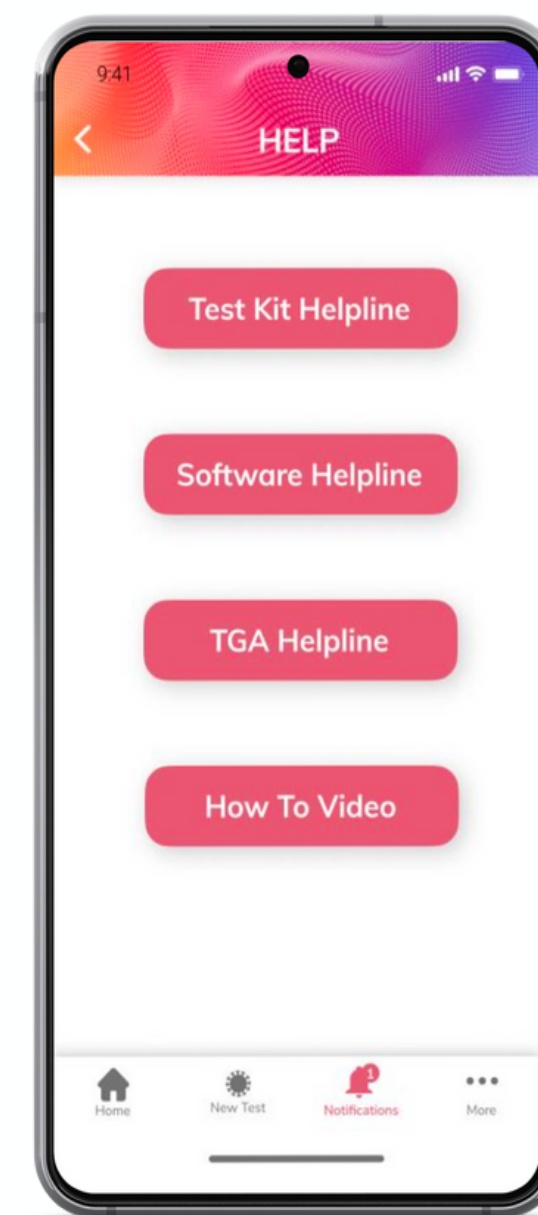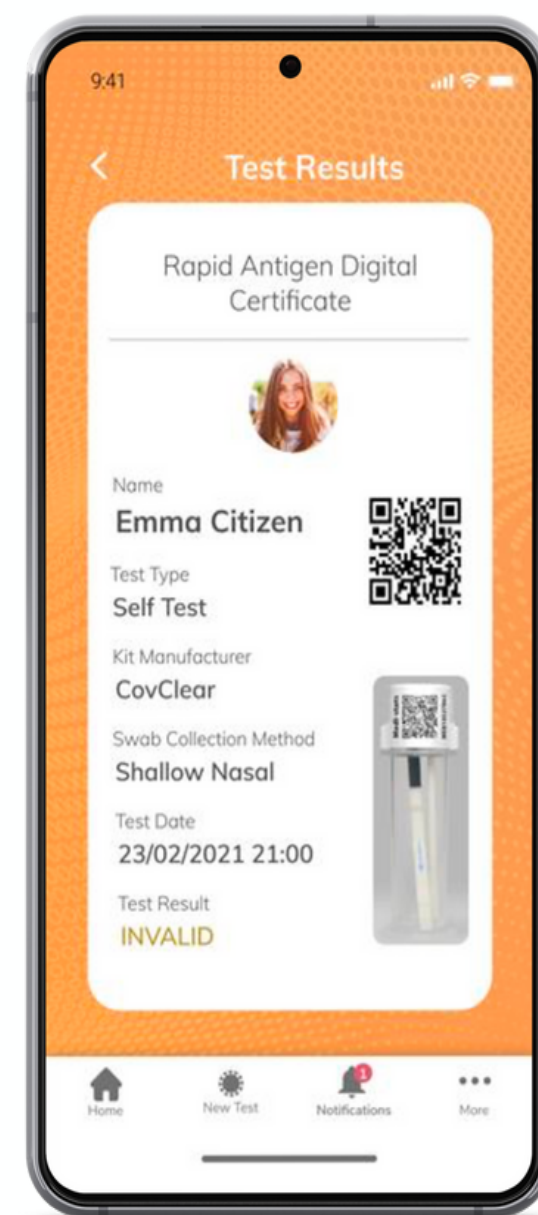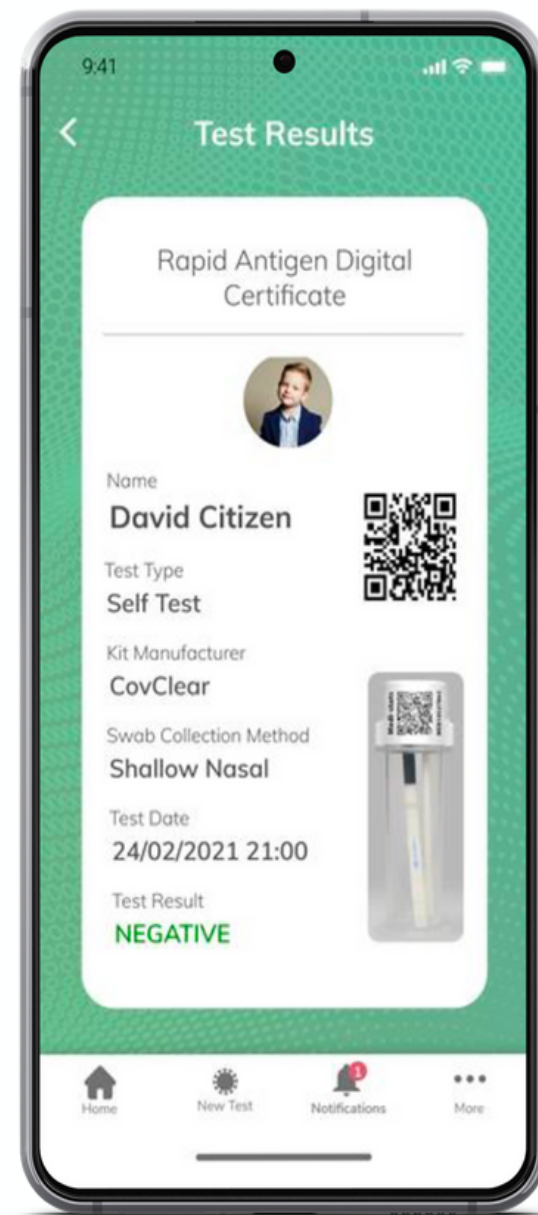

SELF TEST CERTIFICATE AND SUPPORT LINES

The App provides an instant test certificate to enable testers to show they have tested **NEGATIVE**, and are safe to work or enter a venue.

In the event the Test result is '**POSITIVE**' or '**UNCLEAR**' access to support numbers is provided. This is all INCLUDED as part of our service.

User to Test Validation - Unique Test ID

Each HIGHTOP test comes with a **unique ID**.

During the test procedure, the test user is directed to QR Code scan or enter the unique ID number. This links the test to each user and removes the test from the register, so this test ID can never be used again. This guarantees that every **HIGHTOP** test taken via the **Self Check App** can be validated to each test and user.

EVENT TESTING - An attendee can be sent a QR Code to generate a test within 2-3 days of an event. This shares the event test result with the organiser or company. Alternatively, the individual has the option to share their test result with another flow.

For companies also requiring live **TEST SUPERVISION**, a testing event can also be generated and sent, to book in a test event that can take place over video or other.




Management Reporting

Simple dashboards provide early detection of cases

Workers can choose to share their test data with multiple employers.

Easy to read graphics can show percentage of workforce tested, invalid tests and many other metrics essential for safe working and effective business continuity.

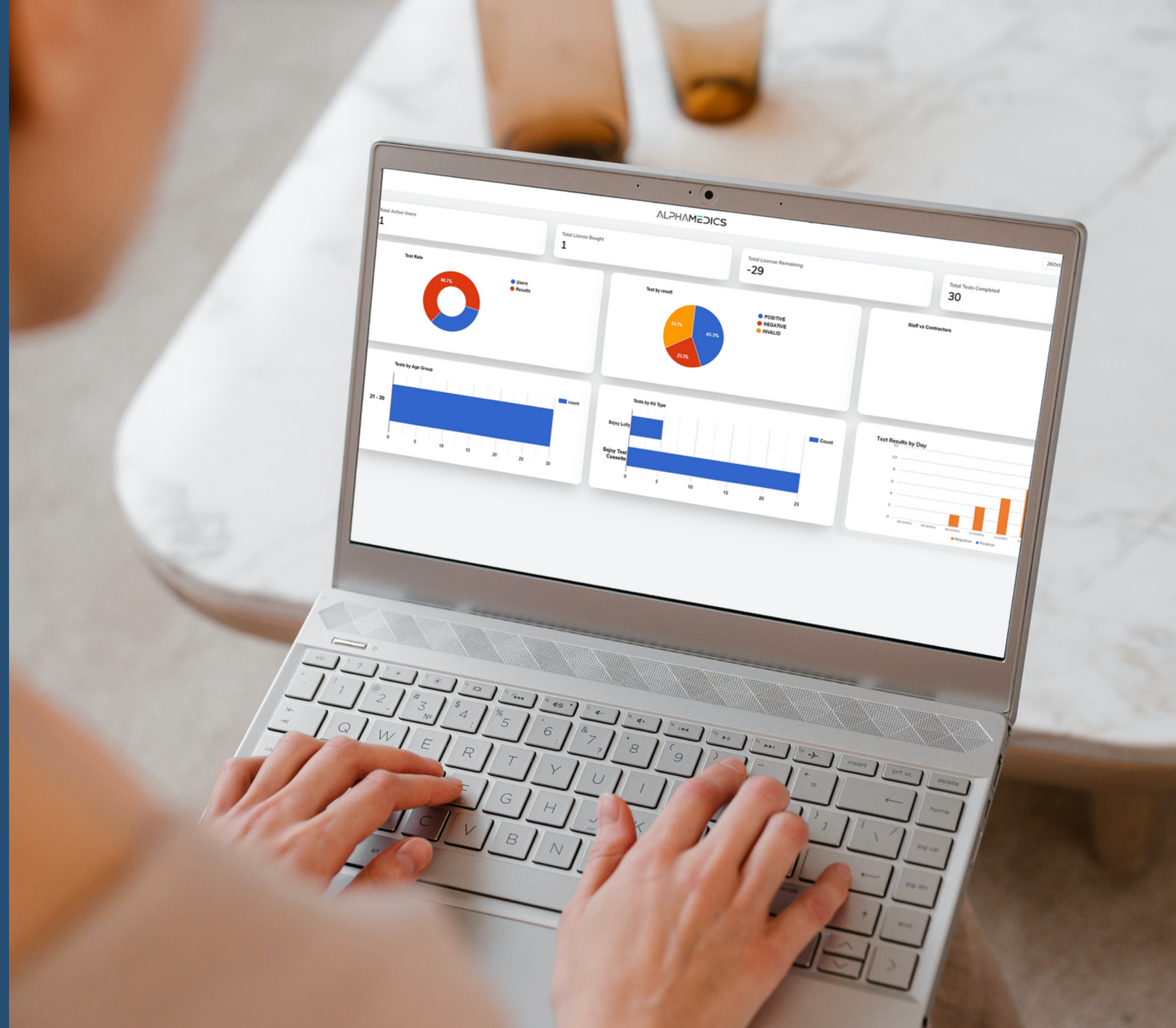



CUSTOMISE YOUR VIEW

Your dashboard can be customised to suit your setting or view just local or global view of your workforce.

Test reports can be downloaded or printed and it's easy to use.

TEST RESULTS & REPORTING

See all test results across your workforce in real-time and downloadable for audit reports.

This single dashboard can keep a track of testing history and provide report exports as necessary.

Event	ALPHAMEDICS							Upcoming Event 20	Active Event
Name	Venue Name	Start Date & Time	End Date & Time	Total Tests	Test Results POS/NEG/INCOMP/REFU/BALANCE	Created By			
		01/09/2021 13:00	31/12/2021 23:00	31	0/27/1/0/3				
		15/07/2021 12:30	18/12/2021 15:30	14	0/14/0/0/0				
		16/08/2021 06:00	16/08/2021 20:00	8	0/8/0/0/0				
		18/08/2021 06:00	18/08/2021 20:00	0	0/0/0/0/0				
		16/08/2021 04:00	16/08/2021 12:00	26	2/22/2/0/0				
		24/07/2021 00:30	31/12/2021 00:30	2	0/2/0/0/0				
		16/08/2021 05:00	16/08/2021 16:00	20	0/18/0/0/2				
		16/08/2021 05:30	16/08/2021 13:30	14	0/13/0/0/1				

Contact

ALPHA-MEDICS AUSTRALIA

133 Market Street,
South Melbourne, VIC 3205

T: 03 7022 6852

M: 1800 575 090

E: sales@alpha-medics.com.au

W: alpha-medics.com.au

ALPHA-MEDICS GmbH

Kepserstrasse 5 85356 Freising
GERMANY

