



SEJOY - Lolly (Saliva) Rapid Antigen Test for Self Use

ARTG ID: 386690



Product Overview



This kit is used for in vitro qualitative determination of SARS-CoV-2 antigens in human saliva samples. It can be used for rapid investigation of suspected COVID-19 cases and can be used as a reconfirmation method for nucleic acid detection in discharged cases.

Intended Use: For self-testing

Type: Saliva Lolly Test



Product Characteristics

Clinical Performance:

- Sensitivity: 93.91%
- Specificity: 99.15%

Advantages:

- Lolly Test, easy to use
- Result only in 10 min
- Suitable for Adults and **Children** 2+ years (with supervision)
- No equipment required



Kit Composition

- SARS-CoV-2 Antigen Lolly Test
- Biohazard Waste Bag
- Instruction for Use

Kit size: 1 Test per box available

Storage: 2-30 Celsius




Specimen Collection

Specimen Collection:

Saliva Sample



Open the aluminum foil bag, take out the cassette.

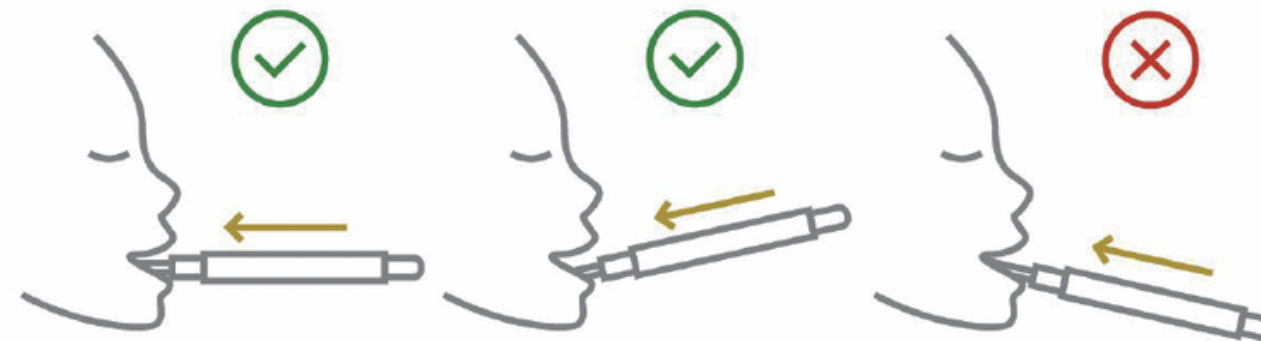

Insert the absorbent tip into the mouth. Make sure the cassette is horizontally placed. Swab the absorbent tip in the mouth and tongue to collect oral fluid.

INCLUDES DIGITAL TEST CERTIFICATE SOFTWARE

Sample Processing:



Take the absorbent tip out from the mouth when the purple color move across the result window.

Start Timer (10min)



Positive



Negative



Invalid